

# Bingo Evolution

## Head Support Speakers



FOR BINGO EVO HEAD SUPPORT PADS

### IMPORTANT CONSUMER INFORMATION NOTICE:

This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

### SUPPLIER:

This manual must be given to the user of this product.

### USER:

Before using this product, read entire manual and save for future reference.

medifab®

User Manual for Bingo Evo Head Support Speakers  
Manual:

Serial Number:

# About the product

The Bingo Evo Stroller has an optional accessory of head support speakers to place inside the head support wings. This is an ideal way to lull your child to sleep or provide the ultimate soothing sensory experience!

**Product code: 2880-8330-000**

**Please note:** The Head Support Speakers require the Bingo Head Support Wings with spacer fabric (2880-447X-000) for use. The X in the code refers to size (Add 1 for size 1; add 2 for size 2)



The Bingo Head Support Speakers also come with an extra set of soft covers and cable ties for tying loose cords to the seat unit of the stroller (not shown in picture on left).



Soft cover with Velcro backing to secure the speaker to the inside of the head support.



Standard 3.5mm jack that will fit the headphones output on most standard mobile phones/mp3 players.

# How to install

Here is a step-by-step process explaining how the Head Support Speakers for the Bingo Evo are fitted to the Head Support Wings.



## Step 1

Remove the head support pads from the seat unit (you can leave the brackets attached to the seat unit) as shown on right..

## Step 2

You will notice that there are three separate foam inserts in each head support pad as shown on right. Remove the insert that is closest to the mesh side of the wing (this is the insert that is closest to the child's ear).



## Step 3

Take the speakers and peel the paper backing off the sticky-sided Velcro and stick these on the foam insert.

## Step 4

Put the foam insert with the attached speaker back into the head support wing making sure that the speaker will be closest to the child's ear (as shown).



## Step 5

Repeat the process for the other head support wing.

## Step 6

Place the head support wings back onto the head support wing brackets.

## Step 7

Tie back the loose cords with the provided cable ties if required.





Distributed by Medifab Ltd

32 Detroit Drive  
Rolleston 7675  
New Zealand

+64 3 307 9790

[solutions@medifab.com](mailto:solutions@medifab.com)

[www.medifab.com](http://www.medifab.com)

A large decorative graphic consisting of a blue-to-purple gradient semi-circle on the left and a white circle on the right, overlapping the semi-circle.

DISTRIBUTED BY: